de	Gebrauchsanweisung	Otoskope
en	User's Manual	Otoscopies
fr	Mode d'emploi	Otoscopies
it	Istruzioni per l'uso	Otoscopi
es	Instrucciones de empleo	Otoscopios
pt	Manual de operação	Otoscópios
ru	Руководство по применению	Отоскопы

KaWe EUROLIGHT®

KaWe COMBILIGHT®

KaWe PICCOLIGHT®

QM-1-132 R


- de** Bitte lesen Sie diese Gebrauchsanweisung vor der Benutzung sorgfältig und vollständig durch und beachten Sie die Pflegehinweise.
- en** Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the given care instructions.
- fr** Lisez ce mode d'emploi attentivement et entièrement avant d'utiliser l'appareil et respectez les consignes d'entretien.
- it** Si prega di leggere attentamente queste istruzioni per l'uso prima di utilizzare lo strumento e di seguire i consigli per la manutenzione.
- es** Por favor, lea con atención las presentes instrucciones de empleo en su totalidad y siga las indicaciones referentes al cuidado del aparato.
- pt** Antes de utilizar este produto pela primeira vez, favor de ler com muita atenção todo este manual de operação e observar as indicações relativas à manutenção.
- ru** Перед использованием следует полностью и внимательно прочитать настоящее руководство по применению и соблюдать указания по уходу!


Deutsch	8-12	
English	13-17	
Français	18-22	
Italiano	23-27	
Español	28-32	
Português	33-37	
Русский	38-42	

EUROLIGHT® / COMBILIGHT®


1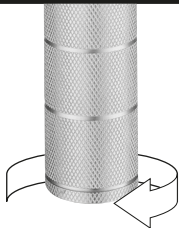

2

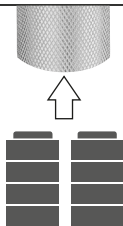3

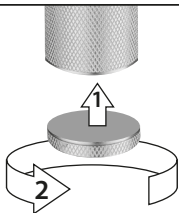
PICCOLIGHT®

1




2

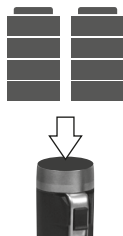

3

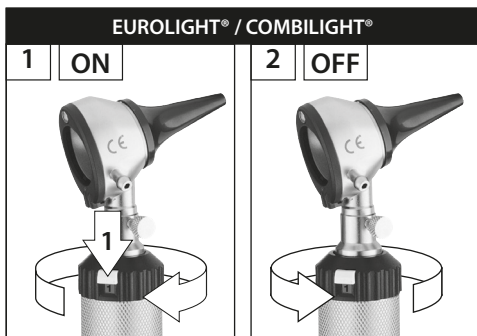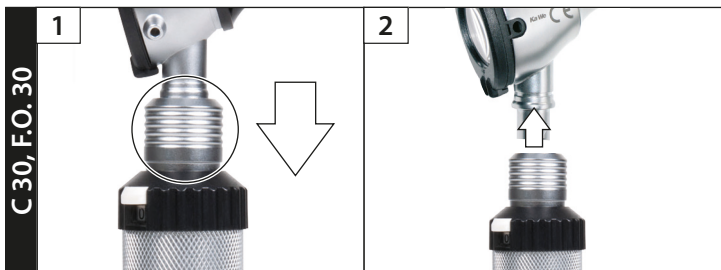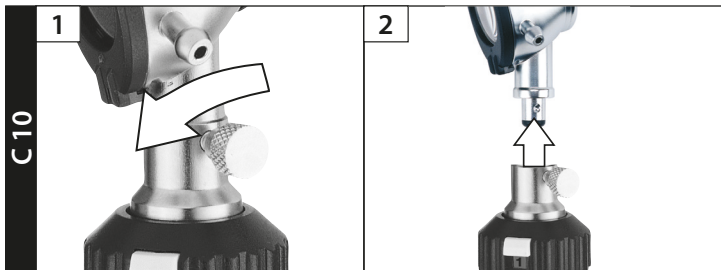Lampenwechsel | Lamp exchange | Échange de la lampe | Cambio della lampadina | Cambio de lámpara | Substituição de ampolas | Замена лампочки

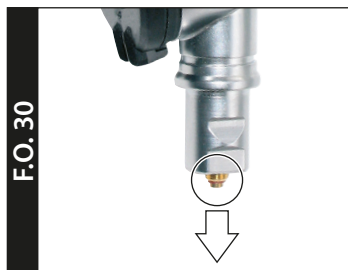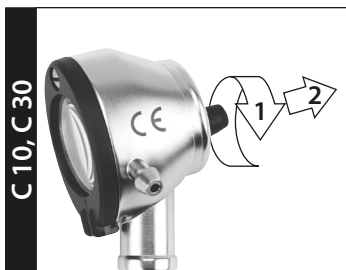
Lampenwechsel | Lamp exchange | Échange de la lampe | Cambio della lampadina | Cambio de lámpara | Substituição de ampolas | Замена лампочки

PICCOLIGHT® C


1


2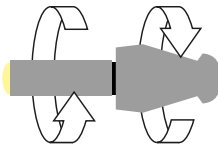


3

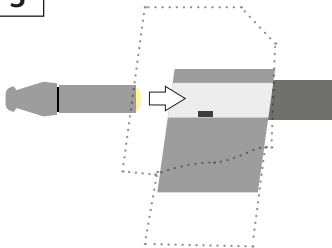
4

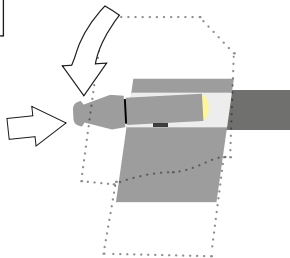5

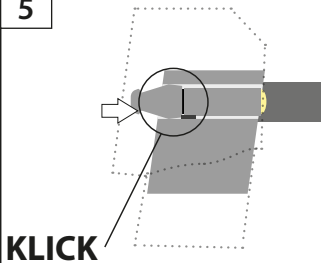PICCOLIGHT® F.O.

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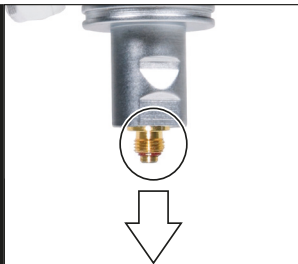Dear Customer, we thank you for choosing a KaWe product. Our products are known for their high quality and longevity. This KaWe product meets EC standards 93/42/EEC (standards for medical products).

Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the given care instructions. Familiarize yourself thoroughly with the operation of this unit before using it.

Retain this User's Manual for future use and pass it on to the next user of this device.

Utilisation:

KaWe otoscopes are only to be used by authorised personnel. They are used for the medical examination of natural bodily orifices such as the ears, nose and mouth.

Purpose:

KaWe otoscopes with attached ear funnels are used for direct examination of the auditory canal and eardrum.


Operation:

KaWe EUROLIGHT®/KaWe COMBILIGHT®: Satisfactory operation is guaranteed with either the use of standard 2.5V KaWe handles with 2 batteries (type C, 1.5V, alkaline batteries recommended, or KaWe 2.5 V or 3.5 V rechargeable batteries (accumulators). Check for correct polarity and for correct bulbs (2.5V and 3.5V)! To attach the the instrument head, slip it on and click it into place or clamp it on.

Turn on the sliding switch by pushing and moving it to the left. The rheostat enables the light intensity (maximum switch position) to be adjusted easily. Turn off the sliding switch by moving it to the right back to its starting

position. Replace the battery by unscrewing the bottom cap. Handles with rechargeable batteries can be charged with the KaWe MedCharge® 4000 (NiMH as well as lithium-ion rechargeable batteries). ☐ ➔ 4,5

KaWe PICCOLIGHT® F.O. + C: Satisfactory operation guaranteed when using KaWe standard 2.5V handles with 2 batteries ("AA" Mignon 1.5V). Turn the unit on by activating the sliding on/off light switch. Replace the battery and bulb by unscrewing and replacing the KaWe PICCOLIGHT® head. ☐ ➔ 4,5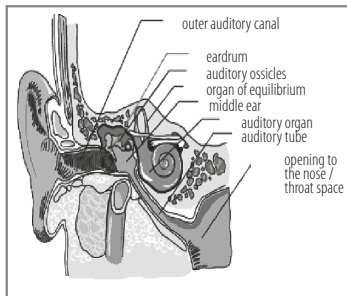

What is the purpose of the general medical otorhinolaryngological examination of the ear?:

This examination enables the doctor to diagnose different diseases and functional disorders in the areas of the outer ear, the auditory canal, the ear drum and the inner ear and to monitor the treatment of these areas.


Lamp exchange:

Satisfactory operation of the unit can only be guaranteed with the use of original KaWe lamps, as only these guarantee adequate light strength and suitable light type.

KaWe PICCOLIGHT® C

Swing the magnifying glass upwards and grasping the adapter, pull out the lamp. Remove the lamp from the adapter, and, if necessary, clean the glass bulb of the new lamp with alcohol. The lamp (along with the adapter) must be pushed in fully until it touches the back of the opening. The glass bulb should be clean and free of grease. ☐ ➔ 7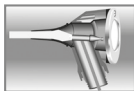

KaWe EUROLIGHT® F.O., KaWe COMBILIGHT® F.O., KaWe PICCOLIGHT® F.O.

Take off the F.O. otoscope head from the handle and remove the lamp by pulling it out. If necessary, clean the glass bulb of the new lamp with alcohol. The glass bulb should be clean and free of grease. The lamp must be pushed in fully until it touches the back of the opening. ☐ ➔ 6,7

KaWe EUROLIGHT® C, KaWe COMBILIGHT® C

Pull out the lamp in the direction indicated by the arrow from the KaWe EUROLIGHT® head and remove the black plastic sleeve from the lamp. Put in the new lamp and then slip the plastic sleeve back onto the lamp. The glass bulb should be clean and free of grease.

☐ ➔ 6

Unsuited use/contraindication of the products:

Any other than the specified clinical diagnostic use described here is not in accordance with the intended use of the unit. The manufacturer is not liable for any resulting damages. There are no known risks. The user alone bears the risk.

Further information, maintenance, storage:

When used and stored properly, these otoscopes will serve you many years. KaWe otoscopes are only to be used with KaWe-brand replacement parts and accessories.

Guarantee:

When used under normal circumstances and with attention to these instructions, we guarantee this product for two years after the date of purchase. (With the exception of lamps/rechargeable batteries/batteries.) Should you need further information, or should your instrument require repair, please contact your dealer.

Warning instructions:

When cleaning, disinfecting and sterilising reusable ear funnels, special attention is required. A temperature of 134°C is not to be exceeded in an autoclave. It is recommended, that the reusable ear funnels be cleaned as soon as possible after their use on a patient.

KaWe otoscopes are to be used at room temperatures of not greater than 35° C. KaWe otoscopes, like all electrically-powered diagnostic instruments, are subject to special safety measures with regard to their electromagnetic compatibility. The operation of the KaWe otoscopes can be influenced by mobile high-frequency communication devices.

Limitation of reprocessing:

Frequent professional cleaning affects the instruments only minimally. The length of the working life of these instruments usually depends on the wear and tear and damage by use.

Instructions

Workstation: Only place the instruments on a clean and sufficiently-sterilised surface.

Storage and transport: No special requirements

Manual cleaning: The exterior surface of the instrument can be cleaned with a damp, soft and lint-free cloth. Never allow the otoscope to come into contact with fluids and ensure that liquids do not penetrate the housing. As the otoscope is not intended for use during operational procedures, simple cleaning and or disinfection with an alcohol-based surface disinfecting agent is sufficient. Do not use abrasive cleaning agents! The unit cannot be sterilised. The magnifying glass is to only be cleaned with glass cleaner.

Automatic cleaning: No automatic cleaning possible/required.

Disinfection: Only reusable ear funnels need to be disinfected with (for example) Helipur. Follow the concentrations and reaction times given by the manufacturer. The otoscope can be disinfected by wiping it with a cloth dampened with (e.g. isopropyl) alcohol.

Sterilisation: Only the reusable funnels can be sterilised. Sterilise them in an autoclave at a temperature of 134°C for a minimum of 3 minutes.

Maintenance: Defective parts must be removed and/or replaced. If the instrument is not functioning, follow these steps: 1. Replace batteries or rechargeable batteries and check the instrument for functionality; 2. Replace the lamp and check once more; 3. Send the unit in for repair.

Check and performance test: Check all movable parts such as the frames of the magnifying glasses and plugin connections to ensure that they all move smoothly. Turn on the unit. For all units: Visually check all parts for damage and wear. Keep all electrical contact points clean.

Packaging: Individually, in a cloth bag or box.

User's Manual/Maintenance Instructions for Rechargeable NiMH/Li-Ion Batteries

Purpose: The rechargeable batteries are to be used as an energy source for the relevant KaWe products.

Further information, maintenance, storage: The rechargeable batteries can be cleaned with a damp cloth and then be wiped off with a dry cloth. Please ensure that no short-circuit is produced.

Further maintenance instructions that may prolong the lifetime of the rechargeable battery:

Please heed the following instructions: The nonobservance of these instructions may result in the battery losing its full capacity within a short period of time. With NiMH technology, a so-called "lazy battery effect" can occur. This can cause the rechargeable battery to discharge too much if there is reduced output voltage. This effect can be nearly eliminated by completely discharging the battery from time to time and then by recharging it two or three times in quick succession. Subsequently, the rechargeable battery will regain its full capacity. If the battery no longer provides sufficient energy to operate the unit, immediately turn off the unit and charge the battery (in order to prevent it from losing its charge completely, which would result in irreversible damage to the battery).

Please note that the rechargeable batteries can spontaneously discharge. If the batteries are not needed (used/charged) for a longer period of time, a self-discharge may occur. The capacity will be lost and the batteries may thus become permanently damaged. The manufacturer is not liable for any resulting damages. When stored for longer than one month, the rechargeable batteries should be partially charged to approx. 50% and high storage temperatures (>30°C) should be avoided.

Please always heed the given charging times in order to avoid overcharging and thus damaging the rechargeable battery. Use the KaWe charging station KaWe MedCharge® 4000, which has an integrated automatic shut off function.

Risks and notice of possible danger:

Avoid short-circuiting the rechargeable batteries.

Never allow the rechargeable batteries to come into contact with liquids.

Keep the rechargeable batteries away from heating installations and open flames.

Rechargeable batteries are not to be immersed in water or salt water

Never try to disassemble or modify the batteries or expose them to heat.

Avoid dropping the batteries and protect them against being hit or crushed as this may damage the housing.

Please only use rechargeable batteries and accessory parts recommended by KaWe.

When using a rechargeable battery for the first time, please make sure to charge it completely.

When charging is completed and the charging unit and the compact adapter are no longer in use, please disconnect them from the unit and the socket in order to avoid fires and other dangers.

Do not cover the charging unit with any kind of material (table clothes, carpets, bedclothes etc.) while operating the instrument. If you use the adapter for a longer period of time, it may become overheated, lose its original shape or catch fire.

Please only charge the rechargeable batteries in the appropriate charging unit. The charging unit and the compact adapter have been developed especially for use with your unit. Please do not use them with other

products or batteries. There is danger of excessive heat buildup, which can cause fires and electrical injuries.

⚠ Prior to disposing of the battery, ensure that the contacts are masked with adhesive strips or other insulating material in order to prevent their direct contact with other materials. Contact with other metallic material in waste bins can lead to fires and explosions.

Charging times:

REF	Capacity [mAh]	Charging time [h]
12.80110.722	3000	approx. 12*
12.80220.722	2200	approx. 10*
12.80120.742	700	approx. 10**

* Charging time in the KaWe charging station KaWe MedCharge® 4000 after complete discharge of the battery.

** Charging time in a wall socket after complete discharge of the battery.

at 30 % - 75% rel. humidity

Storage	0°C to +40°C
Transport	-20°C to +50°C
Operation	+5°C to +40°C

Disposal:

The product is to be disposed of in a manner that is compliant with local ordinances.

Common accessories:

Further information about these articles can be found on our homepage: www.kawemed.de.

Manufacturer:

Kirchner & Wilhelm GmbH & Co. KG
Eberhardstraße 56
DE-71679 Asperg

Contacting the manufacturer:


Address or telephone number of the dealer or dial +49-7141-68188-0.

Accessories (type B):


Disposable ear funnel: 2.5 / 4.0 mm

Reusable ear funnel: 2.5 / 3.5 / 4.0 / 4.5 mm


Installing the batteries on the KaWe EUROLIGHT®, 2 x "C" batteries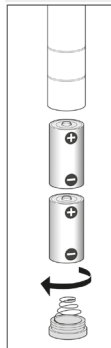

Installing the rechargeable batteries on the KaWe EUROLIGHT® / COMBILLIGHT®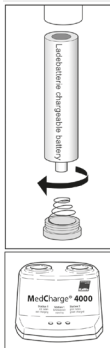

Installing the batteries on the KaWe PICCOLIGHT® 2 x Mignon "AA" batteries

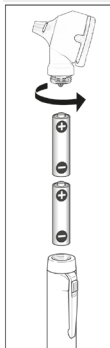Symbol key:

	Manufacturer
	Date of manufacture
	Heed the User's Manual
	Separate disposal of electric and electronic devices
	Dispose of expired batteries separately at an appropriate collection center
	Complies with relevant EU guidelines
	Warning: electromagnetic field
	Only use device in dry rooms
	GOST-R certification for export articles to Russia
	Keep dry
	Temperature limits
	Type B applied part

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KIRCHNER & WILHELM GmbH + Co. KG